nosiboo^{*}go[®]

portable nasal aspirator

Safety instructions

^{*}nosiboo[®]go
portable nasal aspirator

EN

Safety instructions

Dear Customer,

Thank you for purchasing our Nosiboo Go Nasal Aspirator. Please, follow the instructions provided for your device. To obtain the very best performance for your nasal aspirator, please read the **Quick guide** too. For more information we recommend reading the **user manual**:

Please keep these documents for future reference.

Please read all instructions carefully before using the device.

The warranty will be only validated if the device was used according to its intended purpose and does not bear signs of damage.

Nosiboo Go is a hygienic medical device. You may only exercise your right of withdrawal if the device is undamaged and the protective foil on the nose tip is intact. For hygienic reasons, we are unable to refund the product if the foil has been opened.

Safety instructions

Safety signs

Type BF applied parts
(increased protection against electric shock)

Class IIa medical device

Name and address of the manufacturer

Read the Safety instructions

Read the online User manual

For home use only

GTIN – Global Trade Item Number (UDI-DI)

Serial number

Model number

Lot number

Keep dry

Protection against solid particles of diameter greater than 12.5 mm; protected from condensation.

Collect separately the electrical and electronic equipment waste!

Battery

General use

The device is intended to be used only for removing nasal mucus from a child's nose. It shall not be used for any other purposes.

The device cannot be used in oxygen-rich environments and in clinical circumstances! The medical device is for home use only.

Use only as directed. Misuse can be life-threatening.

The device should only be used by adults.

Do not use the nasal aspirator for nasal washing! Aspirating water with the electric device will result in loss of warranty.

In case of unexpected failure or malfunction, immediately switch off the device.

Do not allow children to play with the device, it is NOT A TOY. (The charger cable can wind easily around a child's neck.)

Do not leave the device unattended near children.

Do not use the device in the bathroom or in any other humid or wet environment.

Do not handle the charger with wet hands. Do not cover the vents on the back of the device while in use.

Keep the device and its accessories out of reach of children.

Make sure that all parts are properly attached before using the device.

Do not use the device without the air mesh.

Make sure that the housing and the Penguin head are not damaged or cracked.

Use the device only with the Penguin head supplied by the manufacturer. Use original Nosiboo parts! Our nasal aspirators work effectively only with original Nosiboo parts/accessories.

The bottom of the device should point downwards while in use.

Do not turn the device upside down meanwhile in use.

After use, or if the device's battery just ran low, wait a few minutes until the device cools down and then start charging it.

Cleaning, maintenance and storage

Keep the charger away from heat (e.g. furnace, heater or convector) because it can cause damage to the insulation.

Do not kink the charger, do not place heavy items on the cord, to avoid damage. A damaged cord may cause electric shock.

Do not use the charger if it is cracked or damaged.

Do not remove the mounting screws from the housing. Do not reconstruct the device for other purposes. It may be life-threatening.

When you disassemble the Penguin head for cleaning, do not leave the parts within reach of children. Children may chew or swallow the small parts.

It is not allowed to use a dishwasher, boiling water, microwave oven, vaporizer, or baby bottle sterilizer for cleaning the spare parts, because they might deform by heat!

Do not wash the housing or the charger with water or any other liquid. Do not handle the charger with wet hands. This may cause electric shock or malfunction.

Always unplug the charger from the outlet after charging the device.

The device should be stored in a cool, dry place. Avoid direct sunlight or heat.

Power source

Use the device only with the charger supplied by the manufacturer. Failure to do so may result in fire hazard. Keep the device out of reach of children while being charged.

Protect the charger against damage.

The charger should only be used indoors.

Be sure to charge the device only with 100–240 V AC.

Be sure the charger is compatible with your power outlet!

While being charged, the device cannot be used.

Do not start recharging your device right after it has been charged.

Important information about the battery

The Nosiboo Go devices work with not replaceable Lithium-Polymer (7.4 V, 1500 mAh) batteries.

It is recommended to turn on and charge the device every 3 months to preserve battery performance.

The device should not be disposed of in a household waste bin but should be disposed of as electrical waste in accordance with local regulations. Take it to the designated collection point.

Specifications

During Operations conditions:

1. Temperature range: -5°C to $+40^{\circ}\text{C}$
2. Relative Humidity: max. 75%
3. Atmospheric pressure / air barometric range: 525-795 mmHg / 70-106 kPa.

During Transport conditions:

1. Temperature range: -20°C to $+40^{\circ}\text{C}$
2. Relative Humidity: 10-75%
3. Atmospheric pressure / air barometric range: 150-795 mmHg / 20-106 kPa.

During Storage conditions:

1. Temperature range: -20°C to $+40^{\circ}\text{C}$
2. Relative Humidity: 10-75%
3. Atmospheric pressure / air barometric range: 525-795 mmHg / 70-106 kPa.

Motor performance: 32 W **Differential pressure:** 40 mbar / 40hPa / 4kPa **Max. airflow:** 48 l/min **Power supply:** 100–240 V, 50–60 Hz **Battery:** 7.4 V 1500 mAh Li-Polymer battery pack

DC charger performance: 8.4 V 1 A 8.4 W **Recommended charging time:** approx. 135 minutes **IP class:** IP21 **Protection class:** Internal supply **Type of applied parts:** BF **Applied parts:** the complete housing **Operation type:** no continuous operation **Duration of continuous use:** at least 20 min with fully charged battery **Operation length:** max. 25 min **Housing temperature:** 30–35°C after 1 min of continuous use **Noise level:** 55–62 dB **Expected life of the housing (electrical parts):** 3 years **Expected life of the Penguin head:** 3 months **Diameter of the nasal insertion end of the standard nose tip:** max. 7.0 mm **Length of the nasal insertion end of the standard nose tip:** max. 8.0 mm **Diameter of the nasal insertion end of the newborn's nose tip:** max. 4.9 mm **Length of the nasal insertion end of the newborn's nose tip:** max. 12.6 mm **Weight:** 375 g **Dimensions (housing, W x H x D):** 77 x 190 x 77 mm **Dimensions (complete package, W x H x D):** 95 x 260 x 85 mm

We reserve the right to make technical changes.

Contact

Manufacturer contact details.

ATTRACT Kft.

H-7622 Pécs, Siklósi út 1/1., Hungary

Tel.: +36 72 551 642 (Mo–Fr: 8 am – 4 pm CET)

E-mail: contact@nosiboo.com

Web: www.nosiboo.eu

ATTRACT Kft. is the importer and the responsible person in the European Union for the battery installed in the device.

Contact details of the battery manufacturer:

SHENZHEN MOTOMA POWER CO., LTD., 502, Bldg. N2, Tian An
Cyber Park, FengGang, Dongguan, Guangdong, China,
www.motoma.com, info@motoma.com

PRRC (Person Responsible for Regulatory Compliance):

ATTRACT Kft. – Quality Manager

H-7622 Pécs, Siklósi út 1/1., Hungary

Tel.: +36 72 551 642 (Mo–Fr: 8 am – 4 pm CET)

E-mail: quality@nosiboo.com

Customers in US: www.nosiboo.com

E-mail: customerservice@nosiboousa.com

Phone: (312) 957-6046

ATTRACT Kft. is an ISO 13485 certified medical device manufacturer. This product is an MDD Legacy Device.

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