nosiboo<sup>\*</sup>go<sup>®</sup>  
portable nasal aspirator

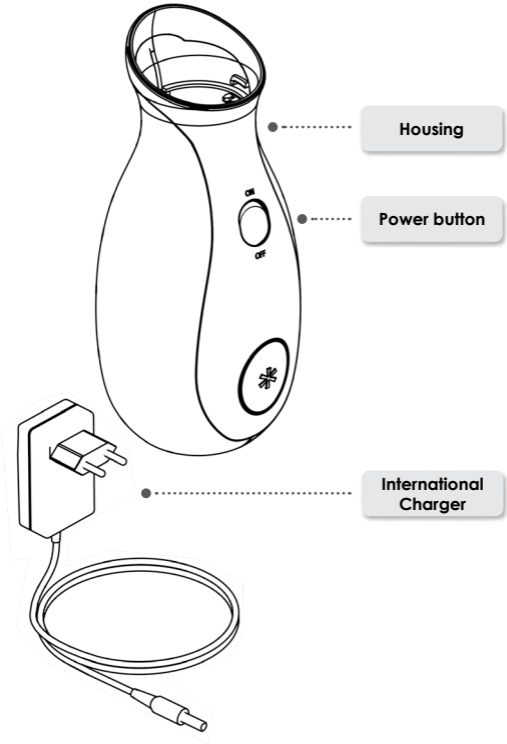
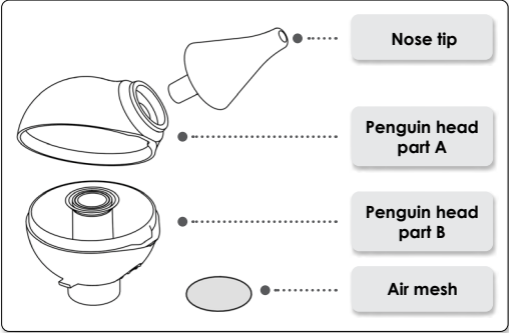
nosiboo<sup>\*</sup>go<sup>®</sup>  
portable nasal aspirator

EN

user manual

# Penguin head

# Foreword

Dear Customer,

Thank you for purchasing our Nosiboo Go Nasal Aspirator. Please read this user manual carefully to obtain the very best performance from your nasal aspirator for years. Please keep this manual for warranty purposes.

## Table of contents

1. Package contents	5
2. Safety instructions	6
3. Intended use	12
4. Before use	13
5. Instructions for use	15
6. Cleaning	17
7. Storage and maintenance	18
8. Troubleshooting guide	20
9. Important information about the battery	20
10. Environmental protection remarks	20
11. Specifications	21
12. Accessories, spare parts	21
13. Malfunction management guide	22
14. International Warranty Certificate	23

Please read all instructions carefully before using the device.

**Nosiboo Go is a hygienic medical device. You may only exercise your right of withdrawal if the device is undamaged and the protective foil on the nose tip is intact. For hygienic reasons, we are unable to replace the product if the foil has been opened.** Guiding and applicable legislation: Directive 2011/83/EU, Section 16 (e) of the European Parliament and of the Commission, Austrian Republic: FAGG: Section 18 (1) 5, Hungary: 45/2014. (II. 26.) Government decree, Section 29 (1) (e). Register your device and we will send you a coupon as a gift that entitles you to a discounted purchase. You may redeem the coupon in the Nosiboo Go manufacturer's online store. [https://product\\_registration.nosiboo.eu/](https://product_registration.nosiboo.eu/)

Attract Kft. is an ISO 13485 certified medical device manufacturer. This product is an MDD Legacy Device.

Actor EUDAMED No.: HU-MF-000004252

## 1. Package contents

Newborn's nose tip

Standard nose tip

Nosiboo Go device

International charger with 3 power plugs

Brush

Pouch with strings

## 2. Safety Instructions

### Safety signs

Type BF applied parts  
(increased protection against electric shock)

Class IIa medical device

Name and address of the manufacturer

Read the Safety instructions

Read the online User manual

For home use only

GTIN – Global Trade Item Number (UDI-DI)

Serial number (Unique device identifier)

Item number (type number)

Lot number (production identifier)

Keep dry

Protection against solid particles of diameter greater than 12.5 mm; protected from condensation.

Hazardous waste

Battery

Warning

DO

Caution

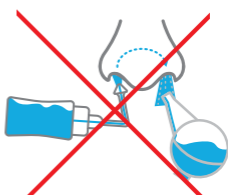
DON'T

**Aspirating water with the electric device  
will result in loss of warranty.**

**Do not aspirate water  
with the device!**

**Do not use the nasal  
aspirator for nasal washing!**

## General use

**Use only as directed. Misuse can be life-threatening.**

**The device should only be used by adults.**

**In case of unexpected failure or malfunction, immediately switch off the device.**

**Do not allow children to play with the device, it is NOT A TOY. (The charger can wind easily around a child's neck.)**

**Do not leave the device unattended near children.**

**Do not use the device in the bathroom or in any other humid or wet environment.**

**Do not handle the charger with wet hands.**

**Do not cover the vents on the back of the device while in use.**

**Use the device only with the charger supplied by the manufacturer. Failure to do so may result in fire hazard.**

**The device is intended to be used only for removing nasal mucus from a child's nose. It shall not be used for any other purposes.**

**Should your baby develop redness of the skin on their nose, please consult your physician, who will advise you about when and how often you should clear your baby's nose.**

**Keep the device and its accessories out of reach of children.**

**Make sure that all parts are properly attached before using the device.**

**Make sure that the housing and the Penguin head are not damaged or cracked.**

**Use the device only with the Penguin head supplied by the manufacturer.**

**The bottom of the device should point downwards while in use.**

**Always switch off the device after use by pushing the power button.**

**Always thoroughly clean and dry the Penguin head after use.**

**Use the brush to clean the nose tip.**

**Read and follow the instructions in the "Important information about the battery" section.**

**Read and follow the instructions in the "Environmental protection remarks" section if you would like to dispose of the device or any part of it.**

**Do not insert the nose tip into the baby's mouth or ears.**

**Do not force or insert the nose tip too far into the nose.**

**Do not press the nose tip against the side of the nostril.**

**Do not use if your child has inflammation or nosebleed. If pain or bleeding occurs, stop using it immediately and consult your doctor.**

**Do not use the device if you see that the Penguin head is damaged. Replace the damaged part. (A list of spare parts can be found in the "Accessories, spare parts" section.)**

**Do not use more than 5 ml of saline solution to dissolve dry and very thick mucus. Excess solution in the Penguin head will exit through the vents on the back of the device.**

**The device should not be used in a clinical or institutional environment. The device is intended for personal use only.**

**Do not attach the Penguin head to other devices.**

**Do not turn the device upside down meanwhile use.**

**Do not use the device without the air mesh.**

**While being charged, the device cannot be used.**

## Cleaning, Maintenance and Storage

**Protect the charger against damage.**

**Keep the charger away from heat (e.g. furnace, heater or convector) because it can cause damage to the insulation.**

**Switch off the device when assembling, disassembling and cleaning the Penguin head.**

**Do not kink the charger, do not place heavy items on the cord, to avoid damage. A damaged cord may cause electric shock.**

**Do not remove the mounting screws from the housing. Do not reconstruct the device for other purposes. It may be life-threatening.**

**When you disassemble the Penguin head for cleaning, do not leave the parts within reach of children. Children may chew or swallow the small parts.**

**It is not allowed to use a dishwasher, boiling water, microwave oven, vaporizer, or baby bottle sterilizer for cleaning the spare parts, because they might deform by heat!**

**Do not wash the housing or the charger with water or any other liquid. Do not handle the charger with wet hands. This may cause electric shock or malfunction.**

**Make sure that all parts are dried thoroughly before the next use.**

**Always unplug the charger from the outlet after charging the device.**

**The device should be stored in a cool, dry place. Avoid direct sunlight or heat.**

## Power source

**Use the device only with the charger supplied by the manufacturer. Failure to do so may result in fire hazard.**

**Keep the device out of reach of children while being charged.**

**Protect the charger against damage.**

**Switch off the device when assembling, disassembling and cleaning the Penguin head.**

**The charger should only be used indoors.**

**Make sure your charger is compatible with your electrical outlet.**

**Do not use the charger if it is cracked or damaged.**

**Do not kink the charger, and do not place heavy items on it, because a damaged cord may cause electric shock.**

**If you have not used your device for a longer period of time, make sure to charge it before use.**

**Always unplug the charger from the outlet when not in use.**

**Even if the device is not used for a longer time, it is worth keeping it in a charged state, and recharging it at least every 3 months to preserve the battery life.**

**While being charged, the device cannot be used.**

### 3. Intended use

The Nosiboo Go Nasal Aspirator is intended for removing nasal mucus from a child's nose. The Penguin head has a patented design that prevents mucus from entering the device.

#### **Intended purpose:**

- \* to ensure nasal breathing for infants
- \* to relieve symptoms of common childhood colds by aspirating nasal mucus
- \* to relieve childhood allergy-like symptoms by removing nasal mucus

#### **Do not use the device in the following cases:**

- \* if the child has hemophilia
- \* if the child has frequent nosebleeds due to dilated vessels in the nasal sept
- \* if the child is taking blood-thinning medicines
- \* in the event of congenital malformations (e.g. cleft palate)

**The device should only be used by adults.**

The device is intended to be used only for removing nasal discharge from a child's nose. It shall not be used for any other purposes.

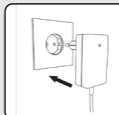
## 4. Before use

### 1. Charging the device


- a) Attach the charging cable to the connector situated at the back of the device housing.

- b) Plug the other end of the international charger into a 100-240V AC outlet. Use the power plug suitable for the outlet.

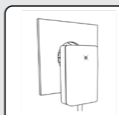Charging is in process if the LED indicator on the charger is red. Charging is complete once the light turns green.


### 2. Assembling the Penguin head


**Use original Nosiboo parts!**  
**Our nasal aspirators work effectively only with original Nosiboo parts/accessories.**

- a) Clean and dry the parts of the Penguin head.

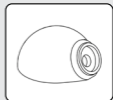Penguin head part A

Penguin head part B

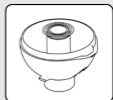
Nose tip

b) Place the two parts of the Penguin head in the correct position.

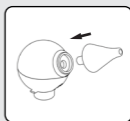c) Push the nose tip into place.


**Newborn's nose tip:  
0-6 months (baby's age)**

**Standard nose tip:  
6 months and older**

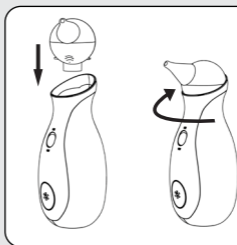**3. Attach the Penguin head to the device housing.**

**4. The device is ready for use.**

**Make sure your charger is compatible with your electrical outlet. Failure to do so may cause electric shock or fire.**

**If you have not used your device for a longer period of time, make sure to charge it before use.**

**Clean the parts of the Penguin head before use.**

**Make sure that all parts are dried thoroughly before use.**

**Do not start recharging your device right after it has been charged.**

**Do not use the device without the air mesh.**

## 5. Instructions for use

### 1. Hold your child in a stable sitting position.

Sit your child onto your lap, hold them in your arms or sit them into an upright position supporting their back with a pillow. You may also sit them in an infant bouncer or infant seat.

### 2. Insert the clean, dry nose tip into the child's nostril.

The bottom of the device should point downwards while in use.

NOTE:

Do not pinch or block the other nostril while using the nasal aspirator.

### 3. Turn on the device with the power button.

NOTE: Any time you are not actively using the device, switch it off.

NOTE: The device turns itself off automatically after 30 minutes of continuous use, but it can be restarted depending on the charge level of the battery.

### 4. Move the device around gently to make sure that you reach all areas of the nostril.

TIP: The suggested length of contact aspiration is 10 seconds per nostril.

TIP: Dry, very thick mucus can be dissolved by a mild saline solution. After using the solution, wait a few minutes, wipe excess discharge with a tissue, then reuse the nasal aspirator as directed.

NOTE: To avoid unsafe use, do not use more than 5 ml of saline solution at a time. The Penguin head can hold up to this amount of liquid safely and prevent mucus from entering the device housing.

**5. Switch off the device after use with the power button.**

NOTE:

Always switch off the device after use.

**TIP: The Nosiboo Go Nasal Aspirator can be used several times a day. Your general aim should be to make sure that your child can breathe peacefully. It is recommended that you clear the child's airways after waking up, before bed time and feeding.**

NOTE: Nose drops and other preparations should be used as directed by your healthcare professional.

**The device should only be used by adults.**

**Do not allow children to play with the device as it is NOT A TOY.**

**Do not cover the vents on the back of the device while in use.**

**Do not use the device in the bathroom or in any other humid or wet environment.**

**Do not handle the charger with wet hands.**

**Make sure that all parts are dried thoroughly before use.**

**Always switch off the device after use with the power button.**

**After use, or if the device's battery just ran low, wait a few minutes until the device cools down and then start charging it.**

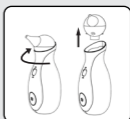
**Do not turn the device upside down meanwhile use.**

**Do not use the device without the air mesh.**

**Do not use more than 5 ml of saline solution. Excess solution will exit through the vents on the back of the device.**

## 6. Cleaning


1. Remove the Penguin head from the device housing, remove the nose tip, then open the Penguin head.

2. Wash the nose tip, and the Penguin head in lukewarm water and dishwashing detergent and let them dry.

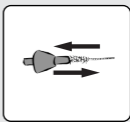3. Use the brush to clean the nose tip.

4. Clean the device housing with a dry, soft cloth; do not use any liquids, detergents or chemicals.

**Switch off the device when assembling, disassembling and cleaning the Penguin head.**

**Do not wash the housing or the charger in water or any other liquid.**

**Do not handle the charger with wet hands, as it might lead to damage or electric shock.**

**Thoroughly clean the Penguin head after use.**

**Make sure that all parts are dried thoroughly before the next use.**

**If liquid gets into the housing due to improper use, make sure that it is completely dry before using it again.**

**Do not disassemble the housing.**

**To avoid damage, make sure the device housing is not exposed to radiant heat.**

## 7. Storage and maintenance

### Storage

1. Store the device possibly in a way that the power button is not pressed to any object, in order to prevent the device from turning itself on.
2. Keep the device in an enclosed space where it is protected from dust and damage (e.g. in a cabinet).

#### NOTE:

Do not store near a heat source.

Do not leave in a vehicle as it can get overheated.

#### NOTE:

The recommended storage temperature for the battery of the device is between 50-86 °F (10-30 °C).

A drop of performance can occur in case of inappropriate storage conditions.

#### NOTE:

It is recommended to turn on and charge the device at least every 3 months to preserve battery performance.

## Maintenance

There is a special air mesh below the Penguin head which prevents solid objects and dust from entering the housing. In the event of a drop in performance, it is recommended that you check the air mesh. If it is dirty or clogged, replace it with a spare air mesh. (A list of spare parts can be found in the "Accessories, spare parts" section.)

**Nosiboo Go Accessory Set Info: [www.nosiboo.eu](http://www.nosiboo.eu)**

**Protect the charger against damage.**

**Keep the charger away from heat.**

**After cleaning, do not leave the parts of the disassembled Penguin head within the reach of children.**

**Do not use a damaged charger, it may cause electric shock.**

**Do not place heavy items on the charger's cable, because a damaged cable may result in fire hazard.**

**Make sure that all parts are dried thoroughly before assembly.**

**Store the device in a cool and dry place. Avoid direct sunlight or heat.**

**If you have not used your device for a longer period of time, make sure to charge it before use.**

## 8. Troubleshooting guide

Malfunction

**The device cannot be switched on.**

Reason

**The device is not charged.**

Recommendation

**Connect the charger to the device, then to a 100–240 V wall socket.**

Malfunction

**The suction intensity is decreased.**

Reason

**The Penguin head is blocked or was incorrectly assembled.**

Recommendation

**Take the Penguin head apart and clean it. After drying the parts of the Penguin head, assemble them according to the user manual.**

Malfunction

**The suction intensity is decreased.**

Reason

**The air mesh has been overused.**

Recommendation

**Replace the air mesh according to the user manual's "Storage and maintenance" section.**

Malfunction

**Liquid or mucus got into the device.**

Reason

**Improper holding of the device or excessive use of saline solution**

Recommendation

**If you experience that mucus or liquid has entered the device then stop using the Nosiboo Go, turn it off and contact the Nosiboo customer service team right away.**

## 9. Important information about the battery

The Nosiboo Go devices work with Lithium-Polymer (7.4 V, 1500 mA) batteries.

The battery can be replaced exclusively by a specialist service.

It is recommended to turn on and charge the device at least every 3 months to preserve battery performance.

After use, or if the device's battery just ran low, wait a few minutes until the device cools down and then start charging it. The battery's 12-month warranty applies only to manufacturing defects, it does not apply to capacity loss.

## 10. Environmental protection remarks

When disposal of the device is necessary, do not throw it in the normal household waste, as electronic products qualify as hazardous waste. This is indicated on the packaging by the following symbol:

*The Penguin head does not present a serious risk to the environment and can be disposed of as household waste.*

# 11. Specifications

## Operating conditions:

1. Temperature range: -5 °C to +40 °C
2. Relative humidity: 45-75%
3. Atmospheric pressure / barometric pressure range: 550-760 mmHg / 73-101 kPa.

## Transportation conditions:

1. Temperature range: -20 °C to +40 °C
2. Relative humidity: 45-75%
3. Atmospheric pressure / barometric pressure range: 150-760 mmHg / 20-101 kPa.

## Storage conditions:

1. Temperature range: -20 °C to +40 °C
2. Relative humidity: 45-75%
3. Atmospheric pressure / barometric pressure range: 550-760 mmHg / 73-101 kPa.

**Motor performance:** 32 W **Differential pressure:** 40 mbar / 40hPa / 4kPa **Max. airflow:** 48 l/min **Power supply:** 100–240 V, 50–60 Hz

**Battery:** 7.4 V 1500 mAh Li-Polymer battery pack **DC charger**

**performance:** 8.4 W **IP class:** IP21 **Protection class:** Internal supply  
**Type of applied parts:** BF **Applied parts:** the complete housing **Operation type:** no continuous operation **Operation length:** max. 25 min **Duration of continuous use:** at least 20 min with fully charged battery **Average housing temperature:** 30–35 °C after 1 min of continuous use **Noise level:** 55–62 dB **Expected life of the housing (electrical parts):** 3 years **Expected life of the Penguin head:** 3 months **Diameter of the nasal insertion end of the standard nose tip:** max. 7.0 mm **Length of the nasal insertion end of the standard nose tip:** max. 8.0 mm **Diameter of the nasal insertion end of the newborn's nose tip:** max. 4.9 mm **Length of the nasal insertion end of the newborn's nose tip:** max. 12.6 mm **Weight:** 375 g **Dimensions (housing, W x H x D):** 77 x 190 x 77 mm **Dimensions (complete package, W x H x D):** 95 x 260 x 85 mm

We reserve the right to make technical changes.

# 12. Accessories, spare parts

## Nosiboo Go Accessory Set:

Accessory pack consisting of Nosiboo Go spare parts

<https://shop.nosiboo.eu/>

- \* 1 Penguin head
- \* 1 newborn's nose tip
- \* 1 standard nose tip
- \* 1 air mesh
- \* 1 brush

NO-02-012-A6-5WCC  
International charger

Parts can be ordered from the Specialist service station:

Attract Kft.

H-7622 Pécs, Siklósi út 1/1., Hungary

Tel.: +36 72 551 642 (Mo–Fr: 8 am – 4 pm)

**E-mail:** [customerservice@nosiboo.com](mailto:customerservice@nosiboo.com)

## 13. Malfunction management guide

*The warranty for the device is valid for 24 months. It will be only validated if the device was used according to its intended purpose and does not bear signs of damage.*

In the next step the existence of malfunctioning has to be clarified, whether the malfunction occurred due to the user's own fault or if it was no fault of theirs. The following checks can be conducted by any user at home or with the distributing merchant or their trustee after presenting the filled in warranty card or the invoice. The legal entity charged with failure assessment is obliged to fill in the warranty card supplied with the User manual or provide their own warranty document, record the steps of troubleshooting in detail and forward it to the manufacturer within 24 hours.

The manufacturer shall not be held responsible for the failure of products from unauthorised import sources!

### What is covered?

The warranty is only valid if the device was used according to its intended purpose and does not bear signs of damage detailed in the "What is not covered" section. The warranty covers malfunctions that are caused by manufacturing defects (problems that were present at the time the device was sold).

#### Issue

**You experience "unusual" issues while operating the device, e.g. the device buzzes or clacks unusually loudly, shakes, or does not turn off after 30 minutes.**

#### Is the warranty valid?

**If the device does not bear traces of physical damage included in "What is not covered," the customer's device is covered by this warranty.**

### What is not covered?

The device is meant for personal use only. Problems that are caused by overuse, abuse, misuse, inappropriate storage conditions are not covered. Also, consequential and incidental damages are not recoverable under this warranty. Furthermore, random and collateral damages and the battery's capacity loss are also not covered by the warranty either.

Issue	Is the warranty valid?
Signs of damage to the label indicate that the device housing has been opened.	Due to misuse of the device, the warranty is not valid.
Signs of external damage to the device housing indicate that the device housing has been opened.	Due to misuse of the device, the warranty is not valid.
Excessive contamination of the air mesh, dirt inside the device housing indicate that the device has been misused.	Due to misuse of the device, the warranty is not valid.
If the power button on the device is broken, stuck, loose or has fallen off, it indicates external impact; therefore, it is not considered to be covered by the warranty. Turning on and operating the device is life-threatening!	Due to misuse of the device, the warranty is not valid.

## Warranty tasks

The distributor of the Nosiboo Go device is obliged to promptly handle customer complaints based on the aspects included in the chapters Troubleshooting guide and Malfunction management guide of the User manual. In case the conditions of warranty apply, the device should be promptly repaired or replaced. The distributor of the device is obliged to keep a written record about the troubleshooting. This can be done within the framework of the distributor's internal documentations or in the warranty card supplied with the User manual. It has to contain the detected malfunction, the date and the unique serial number to be found on the label on the bottom of the device.

## Specialist service station contact details

Attract Kft.  
 H-7622 Pécs, Siklósi út 1/1., Hungary  
 Tel.: +36 72 551 642 (Mo–Fr: 8 am – 4 pm)  
 E-mail: customerservice@nosiboo.com

## 14. International warranty certificate

**Nosiboo Go is a hygienic medical device. You may exercise your right of withdrawal only if the foil packaging of the nose tip is intact! Due to hygienic reasons, we do not replace the device if the packaging has been opened.** Guiding and applicable legislation: Directive 2011/83/EU Section 16 (e) of the European Parliament and of the Commission; Austrian Republic: FAGG: Section 18 (1) 5 Hungary: 45/2014. (II. 26.) Government decree Section 29 (1) (e)

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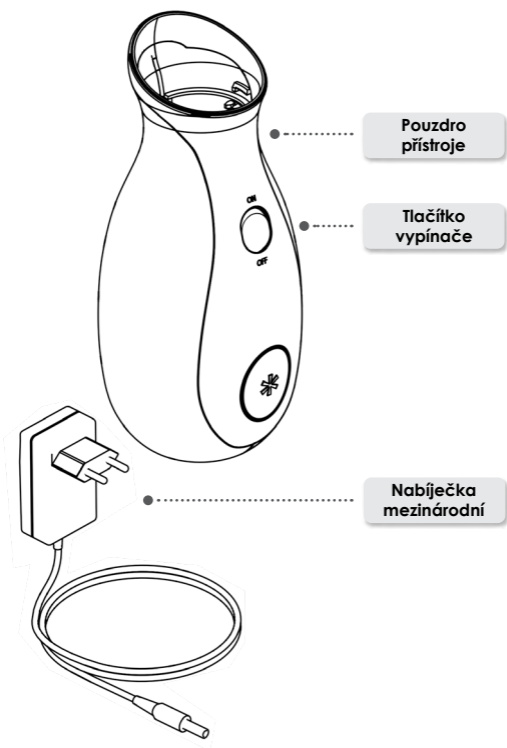
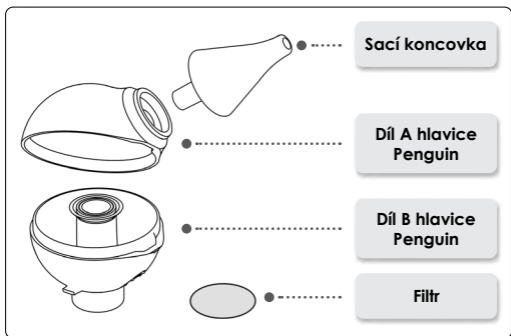
**Nosiboo Go is a hygienic medical device. You may exercise your right of withdrawal only if the foil packaging of the nose tip is intact! Due to hygienic reasons, we do not replace the device if the packaging has been opened.** Guiding and applicable legislation: Directive 2011/83/EU Section 16 (e) of the European Parliament and of the Commission; Austrian Republic: FAGG: Section 18 (1) 5 Hungary: 45/2014. (II. 26.) Government decree Section 29 (1) (e)  
 Warranty card: <https://en.nosiboo.eu/downloads/>

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